



Onsite Instant 7

The Onsite Instant 7 Split Specimen urine drug screening cup is ideally suited to pre-employment testing, random workplace testing and any situation where a comprehensive drug screen is required.

The Onsite Instant 7 is specifically designed to suit **Australian Standard 4308:2008 cut off levels.**

The ideal test choice when screening for 'historical drug use' is required.

Now incorporating K2 test for Synthetic Marijuana.

The Split Specimen function allows the collector to activate the test when he/she is ready, enabling precise timing when interpreting the test strips.

- MET & AMP ON INDIVIDUAL STRIPS
- EASY-TO USE AND CONVENIENT
- CLEAR, RAPID RESULTS IN MINUTES!
- COLOUR CODED TEST STRIPS
- SPLIT SPECIMEN DESIGN ENSURES PRECISE TIMING AND SAMPLE VOLUME

ONSITE INSTANT 7

TESTS FOR THE FOLLOWING:

AMP 300mcg/L (Amphetamine)
MET 300mcg/L (Methamphetamine)
COC 300mcg/L (Cocaine)
OPI 300mcg/L (Opiates)
BZO 200mcg/L (Benzodiazepines)
THC 50mcg/L (Marijuana)
K2 75mcg/L (Synthetic Marijuana)

Plus ADULTERANTS

NIT (Nitrite),
SG (Specific Gravity)
CR (Creatinine)

INCLUDES TEMPERATURE STRIP

Ease-of-collection


Simply unscrew the lid, fill above the marked line and recap ready for reading.

Clear, Rapid Results

After collection, depress the side button to release the urine into the test chamber. Results are clear and easy to read and appear within 5 minutes.

Colour Coded Display

Each drug group is colour coded for ease of recognition.

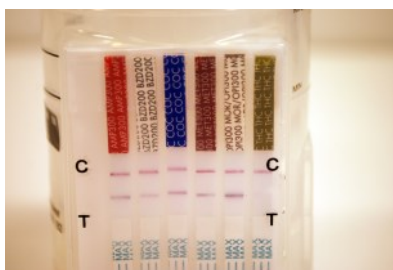

Test Integrity

Integrity of the urine sample is maintained via the temperature test strip and adulterant check.

The temperature test strip will confirm the urine collected is at 'body' temperature.

Adulterant tests will confirm that the urine sample is normal and has not been diluted or contains drug masking agents.

Test control lines confirm the urine sample has flowed correctly and the test is complete.

Comprehensive instructions and procedure cards are included to ensure testing is conducted correctly.



Order Code (Box of 25): OD-073



Australian Designed & AS4308 Verified



Cut-off levels are set for the Australian market reducing discrepancies between initial onsite testing and confirmation testing results. The Onsite Instant 7 is one of a few cups verified to meet AS/NZS4308:2008 and issued with a Certificate of Compliance.

Result Confirmation

Any presumptive-positive results should be confirmed via GC/MS testing using a second urine sample.

Quality Controls

Test control lines use colloidal gold technology with superior sensitivity, accuracy and precision. Cross-reactivity is pretested against most drugs and other substances.

Custom Test Kits

Test kits can be custom designed to suit all workplace applications, incorporating up to 12 drugs in one urine test cup.