

Onsite Eco 7 DFWDLQHUTest Procedure

Allow the test device to reach room temperature (15-30 degrees celcius) prior to testing.

PREPARATION

Bring the pouch to room temperature before opening it. Check the expiry date is still valid. Once you remove the test from the pouch it must be used within 8 hours.

STEP 1

Remove the drug test cup and disposable gloves. Label the device with the donor ID and date.



STEP 2

Hand the cup and gloves to the donor. Advise the donor to urinate directly into the test cup and be sure to fill it between the minimum 30ml and maximum 70ml, as marked on the cup. Then replace the lid securely and return to collector.


STEP 3

Check TEMP strip within 4 minutes of specimen collection.



STEP 4

Peel off the red label to reveal the test results.

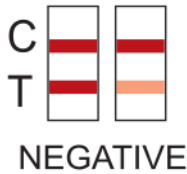

Negative results— Indicated by the presence of 2 lines. No matter how dark/light

Presumptive positive— only the top 'control' line appears

Invalid— the control line fails to appear, usually due to too little oral fluid collected.



STEP 2

Press the vacutainer tube closure cap downwards onto the needle to pierce the tube seal.

The urine will immediately be suctioned into the vacutainer tube to the level designated by the tube.



Training is available to ensure onsite drug & alcohol screening personnel are correctly trained to use and administer drug & alcohol test equipment under **Australian Standards 4308:2008, 4760:2006 & AS3547**. This course has been developed ERI to assist clients in adhering to new legislative changes. Available via classroom and distance learning. For further details please see our training page.